

---

# STONE DEFLECTOR

INSTRUCTIONS FOR MOUNTING AND ASSEMBLING



---

STEALTH PRODUCTS



---

## INSTRUCTIONS FOR ASSEMBLING TONE DEFLECTOR

- 1) Disassemble the Tone Deflector before installing to the Headrest.
- 2) Fasten the Tone Deflector Adaptor to the rear of the headrest using the Socket Head Screws provided. (Four 10-24 x 3/8" or Three 10-32 x 3/8)
- 3) Fasten the Tone Deflector Shock Absorbers to the threaded holes in the Tone Deflector Adaptor Plate.
- 4) Place the Tone Deflector Cover over the Tone Deflector Shock Absorbers(The countersunk holes in the TD Cover will line up with each threaded hole in the Tone Deflectors)
- 5) Insert the M6 x 1 x 10mm Flat Head Screws into each of the countersunk holes in the Tone Deflector Cover. Tighten with 5/32 Hex Key.



**Notes:**                   \*TD100 - Must use the 10-24 x 9/16 Socket Head Screws provided for mounting the headrest to the TD100, using shorter screws will cause stripping of the threaded screw holes.

                                  TD101 - Uses the standard 10-32 x 7/8" Socket Head Screws for mounting the headrest to the TD101

---

## OPERATING AND ADJUSTING RLSHW100 RESIDUAL LIMB SUPPORT



### TD101

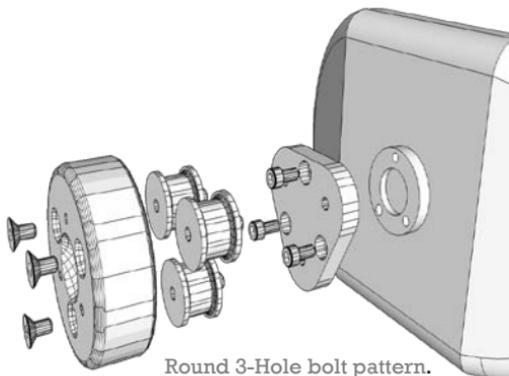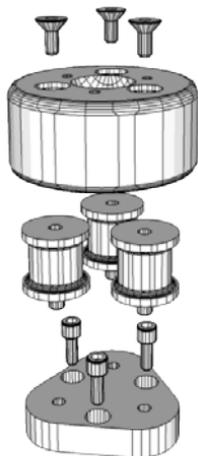

**Tone Deflector for round 3-hole bolt pattern.**

(Comfort Plus, Tri Comfort Plus, Otto-Bock Series)

### TD100

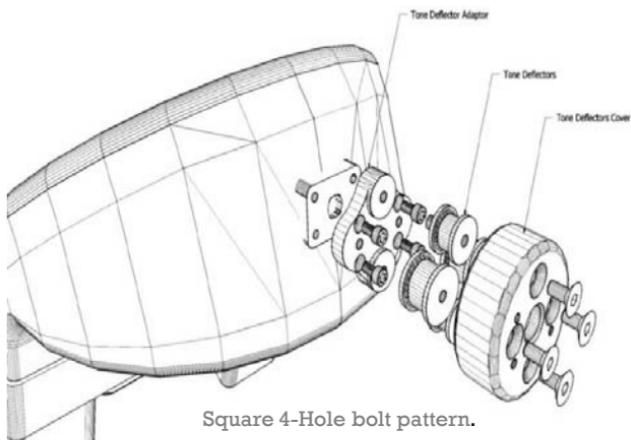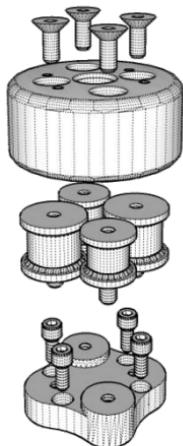

**Tone Deflector for square 4-hole bolt pattern.**

(Stealth Ultra, i2i, All Positioning, Combo Series)

---

## Stealth Products, Inc. Warranty :

**Caution:** These products are designed to be fitted, applied and installed by a healthcare professional trained for these purposes. The fitting, application and installation by a non qualified individual could result in serious injury.

Stealth Products, Inc. warrants this product against failure due to defective materials or workmanship as follows:

Note: SURING bracket welded to head plate is covered under hardware warranty

Covers	180 days
Headrest	180 days
Hardware	5 years

### Limitations:

Stealth Products, Inc. does not warrant damage due to but not limited to:

- Abuse of product
- Misuse or misapplication of products
- Modification of product without written approval from Stealth Products, Inc
- Any alteration or lack of Serial # where applicable will automatically void this warranty
- Stealth Products, Inc. is liable for replacement parts only
- Stealth Products, Inc. is not liable for any incurred labor costs



# Stealth Products

Stealth Products, Inc.

104 John Kelly Dr.

Burnet, TX 78611

Phone: 800-965-9225

Fax: 800-806-1225

Email: [Info@StealthProducts.com](mailto:Info@StealthProducts.com)

---